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LED-A Dental Operating LED Light

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Instruction for use

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4、When the light is off, slide the switch to turn the light on, and the lamp will be turned off if you slide the switch again. When the light is off, you can activate it by a quick sweep of the hand under the sensor switch; sweep again, you will turn the light off (the induction distance is 0-100mm).

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To clean the lamp, please use three-way syringe or something like soft cotton flannel or wet wipes.

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Do not wipe or clean the surface of the lamp with corrosive liquids

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Foshan Huachuang Medical Science And Technology Co.,Ltd

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Acdd: 4th Floor, Building A3, Guangdong New Light Source Industrial Base, Luo Village, Shishan Town, Nanhai District, Foshan City

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Read the manual carefully before using the light

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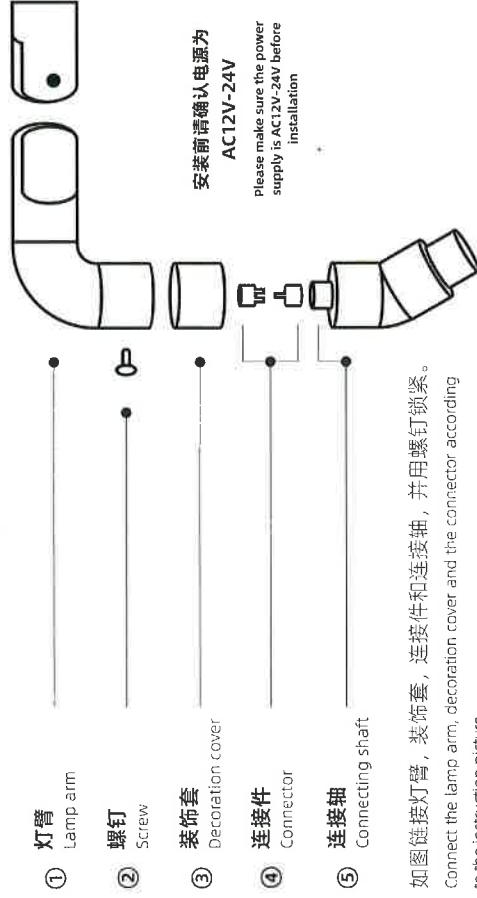
储存和运输环境 / Transportation And Storage Condition

运输温度 / Transportation Temperature	-40-70°C
相对湿度 / Relative Humidity	-10-+100%
大气压强 / Atmospheric Pressure	50-106kpa

五、技术参数 / Technical Data

输入电压 / Input voltage	AC12V-24V
功率 / Power	11W
照度 / Illumination	7000lx / 8000lux
色温 / Color Temperature	4800K-5500K
净重 / Net weight	1146g
毛重 / Gross weight	1700g

三、安装方法 / Installation method

如图链接灯臂，装饰套，连接件和连接轴，并用螺钉锁紧。
Connect the lamp arm, decoration cover and the connector according to the instruction picture.

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1. Loosen the screw according to the picture on the right, adjust the distance and angle to your accustomed position and tighten the screw.

2. When the light is off, slide the switch to turn the light on, and the lamp will be turned off if you slide the switch again. When the light is off, you can activate it by a quick sweep of the hand under the sensor switch; sweep again, you will turn the light off (the induction distance is 0-100mm).